



Extra stability for
school chairs

meru
disabilities to possibilities



Instruction

The MERU Rokzi system provides support and confidence to children sitting in ordinary plastic school chairs with tubular metal legs. Rokzi arms and legs simply clamp onto the metal chair legs, providing the advantage of armrests and anti-tip supports.


New improve design and style, the confidence you need.

Rokzi Arms

- Loosen both bolts from the clamp and place around the rear leg of the chair, positioning as shown.
- Adjust the armrest height by moving the pole through the clamp. Tighen both bolts to set.
- Ensure that the pole is flat against the chair to avoid finger traps.




Mounting Adjustment

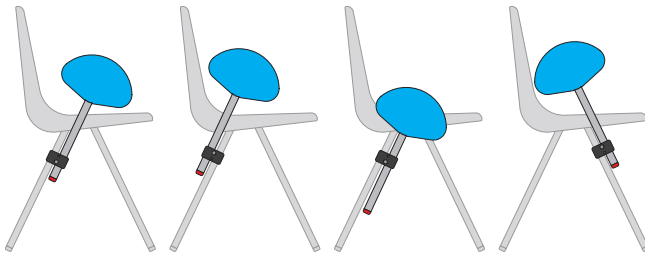

The Rokzi clamps mounts on either leg side, allowing vertical movement for height adjustment. It can be mounted on either the front or rear leg.

Rokzi Legs

- Whilst on a level surface Loosen both bolts from the clamp and place around the rear leg of the chair.
- Position the leg so that it touches the floor, pointing out and away from the chair.
- Tighen both bolts to set.

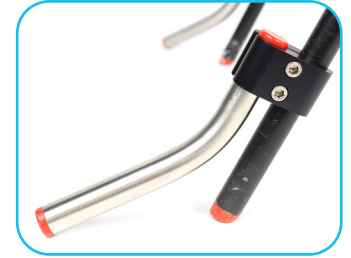

Precautions

- Not suitable for children over 50kg.
- The Rokzi should only be used on the recommendation of a qualified health professional and installed by a competent person
- The Rokzi arms should not be exposed to naked flame or heat sources.
- Unsecured parts of the Rokzi system (the armrests, clamps, legs etc) should not be left around where children could pick them up.
- Children should be in a supervised environment while using A Rokzi.

Important: Rokzi are only suitable for chairs with 19mm (3/4 inch) diameter legs. Do not attempt to fit to chair legs of any other size

About MERU

MERU is a charity that exists to improve life for children and young people with disabilities. To help fund our charitable work we produce and sell products such as Rokzi. **100% of the profits go to MERU charitable activities!**

Warrenty

Each Rokzi is warranted by MERU to be free from defects in materials and workmanship for a period of 12 months from the date of delivery. If any defect is found within this period, MERU will repair, or at its discretion, replace the device without charge for materials and labour. This Warranty is subject to the proviso that the device:

- has been correctly installed
- has been used solely in accordance with these instructions for the purpose for which it was intended
- has not been subjected to misuse or accident, or been modified or repaired by any person other than someone authorised by MERU.

This warranty does not affect your statutory rights.

1 Metcalfe Ave, Carshalton SM5 4AW

+44 (0)1372 725203 | info@meru.org.uk | www.meru.org.uk

